



Acknowledgments	2
Vital Information and Warnings	3
Seating Support General Overview of the Product	4
Installation on a Seat/Chair	5
Installation as a Floor Seating Support	6
Installation of Supplementary Accessories	7
Seating the User	8
Maintenance and Cleaning	9



## **Thank you for choosing Testa-Seat!**

This user guide is designed to ensure you get the most out of our product safely and effectively.

### **Important Safety and Use Instructions**

**Proper Use:** This system must be used and installed as described in these instructions. Incorrect installation or use can endanger the user.

**Intended Purpose:** Use this product only for its intended purpose as described in the user guide. Misuse of the product could lead to damage or injury.

**Regular Maintenance:** Regular checks and maintenance of the seating system are crucial. Refer to the maintenance section of this guide for detailed instructions.

**Disclaimer of Liability:** The company is not liable for any damage or injury resulting from improper use, installation, or failure to follow maintenance guidelines, except as provided by mandatory laws.

**Contact Information:** If you have any questions or require assistance, please contact our customer support at [info@testa-seat.com](mailto:info@testa-seat.com).

**Emergency Procedures:** In case of an emergency or unexpected operation of the product, refer to the emergency procedures section in this guide.

# Custom Seating systems

## Purchase conditions

In collaboration with Occupational Therapists, Physical Therapists, and insightful feedback from parents, we are proud to introduce a new benchmark in comfort and support with our custom seating systems. These systems have been thoughtfully designed for individuals who require additional postural support, making them an ideal choice for a variety of activities involving food and highchairs.

### Key Features:

- **Customized for Comfort:** Each seat is tailored to provide optimum support, catering to the unique needs of each user.
- **Lightweight and Portable:** Our seats are designed for ease of transportation and can be seamlessly attached to various chair types.
- **Multi functional:** Can be used a floor activity chair and like a high chair

### Quality and Variance Disclaimer:

While we are committed to the highest quality standards, please note that due to the custom nature of our seating systems, there may be occasional variances in manufacturing. These variances will not compromise the safety or functionality of the product.

### User Responsibility and Safety Guidelines:

- **Functional Assurance:** Users are responsible for ensuring the product's functionality. Should any issues arise, we urge you to discontinue use immediately and contact us for assistance.
- **Intended Use:** These seating systems are not designed for use as a substitute for a safety car seat, for prolonged sitting periods, or as a substitute for a stroller or carriage.

### Important Warnings:

- **Modifications Prohibited:** Do not make any changes or additions to the seating system or its accessories.
- **Installation Requirement:** Always install the seating system according to the 'User Guide' instructions to ensure maximum safety.
- **Supervision Necessary:** Never leave the user unattended with the system.
- **Not a Toy:** This product is exclusively for seating purposes and should not be used as a toy.
- **Designated Use:** The system is intended for seating purposes only and should not be used for any other activities.

Full Name: \_\_\_\_\_

Signature: \_\_\_\_\_

## Components of the System

1. Removable Padding
2. Upper Anchoring Strap: For securing the seating system to the back of the chair.
3. User Safety Straps.
4. Seat Unit (Pommel): Includes a leg separator, an internal angle, and a universal connector.
5. Lower Anchoring Strap: For securing the support to the seat of the chair.
6. Universal Connector.





## Installation on chair



Placement: The system should be installed on a wide and stable seat whose center of gravity is not affected by the installation of the support.

### Attaching the Straps:

- Align the front of the seat with the front of the chair allowing 90 Deg angle for legs.
- Connect and tighten the lower anchoring strap to the seat of the chair.
- Connect and tighten the upper anchoring strap to the back of the chair.
- After the initial connection, tighten the lower anchoring strap well, and then tighten the upper anchoring strap.

### Warning:

- After installing the system, check that all straps are well-tensioned.
  - Ensure that the system is securely connected and does not sway. To test stability, hold the chair with one hand and try to move the system with the other hand.
- Gap Filling: If a gap forms between the support and the back of the chair, fill the space with a cushion to prevent tipping.

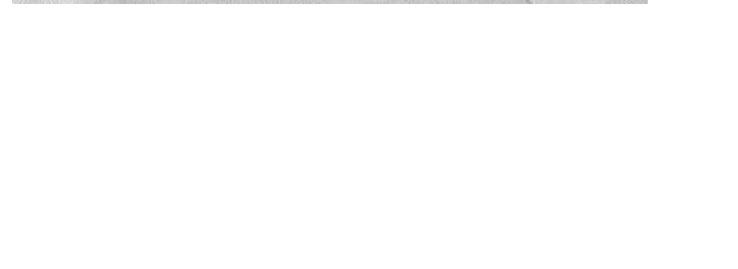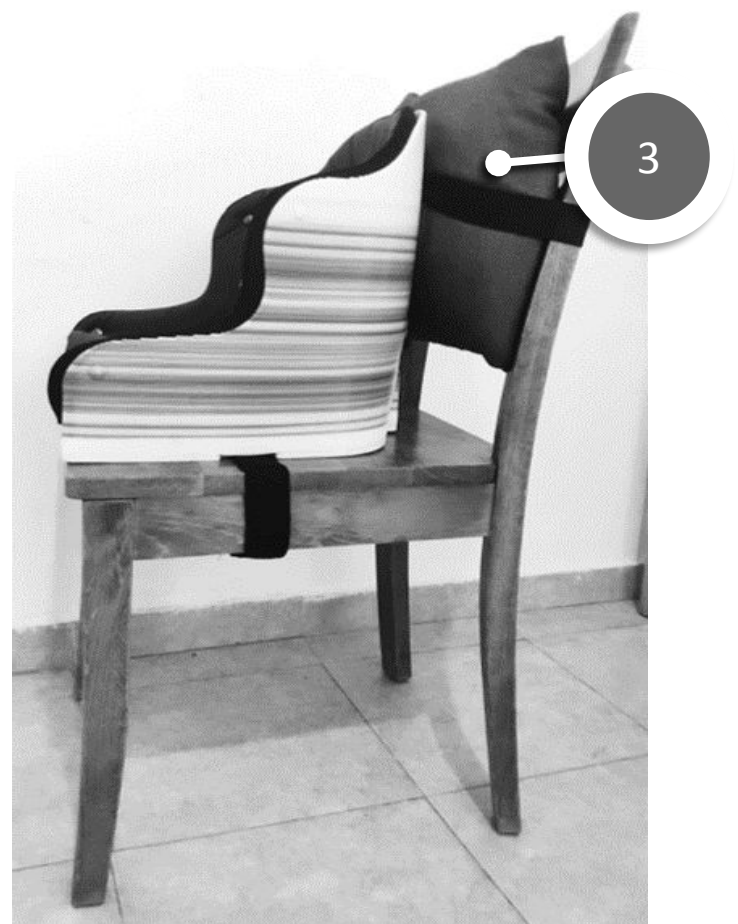

## Floor Placement and Anti-Tipping Installation

Place the system on the floor, ensuring that there are no objects interfering between the system and the floor.

### Connecting the Dedicated Anti-Tipping Device:

- The dedicated anti-tipping device should be connected using a specific "key."
- After assembly, tighten the key securely.

### Warning:

- The anti-tipping device must be installed before seating the user in the system.
- If there is a risk of the user tipping sideways, use a wider dedicated anti-tipping device.



## Installation of Supplementary Accessories

### 1. Head Support:

- Install the head support when the seating support is in place.
- Attach the head support before seating the user.
- Use specific "keys" to connect the back of the support to the rail, and the head support to the rail.
- Tighten the keys well to prevent wobbling.
- Warning: While adjusting the head support to the desired height, hold the user's head until the installation and securing of the head support is complete.

### 2. Leg Rest:

- Install the leg rest when the seating system is in place.
- Attach the leg rest before seating the user.
- Use specific "keys" to connect the leg rest to the rail, and then attach the rail to the body of the system using the front connector.
- Tighten the keys well to prevent wobbling.

### 3. Rear Anti-Tipping Device (Floor Seating):

- See the "Installation as a Floor Seating System" page in this booklet.

### Warning:

- Install all equipment before seating the user.
- Size adjustments should be made when the seating system is installed and stable.
- Ensure that the rail for attaching accessories is long enough.
- Ensure that the keys are tightened sufficiently to prevent "wobbling."
- Before any installation, ensure that the connection is correct and complete.





## Positioning the User

Securing the User in the Chair with the Seating System's Safety Straps:

- Seat the user fully into the seating system, ensuring that their back and buttocks are close to the back of the system.
- Secure the user with the safety straps (anchoring and chest straps).

Warning:

- Only seat users in the seating system for whom the system has been tested and found suitable.
- Do not leave the system and the user unattended.
- Ensure that the safety straps are not loose.
- Dress the user in appropriate clothing that does not interfere with the securing of the safety straps.





## Maintenance and Cleaning

### System's Body

Maintenance and Cleaning Instructions for the Body of the System (ABS Plastic Frame):

- Clean the body of the system using a sponge or cloth with lukewarm water and mild soap only.
- It can be dried with a towel.
- Do not use abrasive cleaning agents; do not use substances based on acetone.
- Do not leave the product exposed to the sun for an extended period.

### Padding

Maintenance and Cleaning Instructions for the Padding:

- To remove dirt, dust, and sand, use a vacuum cleaner.
- To remove a fresh stain, quickly use a damp cloth before the stain dries.
- Do not use bleaching agents, ammonia, or acids.
- Washing Instructions:

DO NOT bleach



Hand wash with mild soap



DO NOT dry clean



line dry



DO NOT machine dry



DO NOT iron

